

## **Introduction**

Congratulations on your purchase of the Cefndy Healthcare X399 Bariatric Toileting Shower Chair. Please ensure you read and understand these instructions in full before construction and use. Please retain these for future reference.

The chair features a soft feel polyurethane moulded seat and upholstery for comfort, warmth and durability. The seat features an integrated clip-on clip-off function for easy cleaning. The arms swing away for easy lateral transfer. The chair is multi use and can be used as a shower chair, commode chair or over a toilet pedestal. The chair has 4 substantial 5" braked castors.

<b>Specifications</b>	
<b>Seat size (W x D)</b>	640 x 460mm 25¼ x 18"
<b>Seat Height</b>	585mm 21"
<b>Width Between Arms</b>	555mm 21¾"
<b>Footprint</b>	670 x 980mm 26¼ x 38½"
<b>Maximum User Weight</b>	254kg 40st



## **Pack Contents**

- Bariatric Toileting/shower chair
- Oval Potty
- User instructions (this leaflet)

## **Assembly Instructions/ Correct Use**

### **Please note:**

- When unpacking carefully examine the product to ensure it is complete. See contents list above.
- Before you begin assembly check for any damage. If damaged please contact your supplier and refrain from using the product.
- **Do not attempt to use if the product is damaged.**

### **Arms:**

- The arms can be rotated to aid lateral transfer by lifting slightly and swinging away from the seat area towards the rear of the frame.

**Footrests:**

- On each side of the front legs of the commode main frame there are two retaining lugs either side.
- Take the footrests and align the holes in the inverted “U” section with the lugs.
- Lower down into place.
- Rotate the foot rest towards the commode and clip into place.
- The footplates can be rotated up and down as necessary, should they temporarily not be required without the need to remove the footrest.

**Handles:**

- The handles can be fitted in two ways either turned inwards to run parallel with the chair back or facing outwards at 90° to the chair back.
- To change push out the pin-clip securing them and rotate handles to suit. Push the clip back in ensuring it is secure.

**Seat:**

- The seat is reversible (the insert can be located to either the front or the rear). This is achieved by lifting away from the location points and rotating 180 degrees before clipping the seat back into location by gently pushing back in to location points.
- The insert can be removed for showering but must be removed for toileting.

**Potty:**

- Empty the potty after each use.
- The potty can be located from either the front or the rear

**Castors:**

- To lock: push down the lever on the castor body (all 4 are lockable).
- To unlock: push the lever in the opposite way.

**Accessories**

A Lap Strap is available for use with the chair. This is available as a separate item from Cefndy Healthcare. It is **essential** that a risk assessment is carried out for each individual user of the Lap Strap. Please refer to the User Instructions supplied with the strap.



## **Cleaning**

---

- The Product can be cleaned using a mild detergent and a mild disinfectant or antibacterial solution caution should be taken when using cleaning agents/chemicals in understanding their suitability.
- Do not use an abrasive cleaner as this may damage the frame and invalidate your warranty. The frame is suitable for mechanical cleaning/disinfection in industrial cleaning machines up to 80°C.
- Thorough drying of the chair is necessary afterwards.

## **Maintenance Requirements**

---

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.
- Please ensure the product is dried following use.

## **Warning**

---

- We recommend a full risk assessment is carried out before use. This should include consideration for the equipment's intended use, the capacity of the user to safely use the equipment, the environment in which the equipment is intended to be used and **all** other relevant considerations.
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.

## **Warranty**

---

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

## **Disposal Instructions**

---

- Appropriate hygiene standards should be applied before disposal to minimise the risk of infection.
- Materials used are essentially non-hazardous and can be disposed of appropriately consideration should be given to local re-cycling arrangements.

## **Contact Details**

---

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare  
Cefndy Road  
Rhyl  
Denbighshire  
LL18 2HG  
Tel: 0044 1745 343877  
E-mail: [cefndy.sales@denbighshire.gov.uk](mailto:cefndy.sales@denbighshire.gov.uk)