

Introduction


Congratulations on your purchase from the Cefndy Healthcare Attendant/self-Propelled Shower Commode Chair range. Please ensure you read and understand these instructions in full before use. Please retain these for future reference.

The chairs feature a soft feel polyurethane moulded seat and upholstery for comfort, warmth and durability. The seat have an integrated clip-on clip-off function for easy cleaning. The arms flip up for easy lateral transfer and features fully adjustable swing away footrests. The chair is multi use and can be used as a shower chair, commode chair or over a toilet pedestal. The chairs have 4 aluminium 5" braked castors.

	T210	T212	MT210	MT212
Seat size	460 x 460mm	460 x 460mm	460 x 560mm	460 x 560mm
Seat Height	500 – 550mm	500 – 550mm	500 – 550mm	500 – 550mm
Total Width x Depth	560 x 860mm	660 x 1030mm	660 x 860mm	760 x 1030mm
Width Between Arms	445mm	445mm	570mm	570mm
Max User Weight	125kg / 19st	125kg / 19st	127kg / 20st	127kg / 20st



Pack Contents

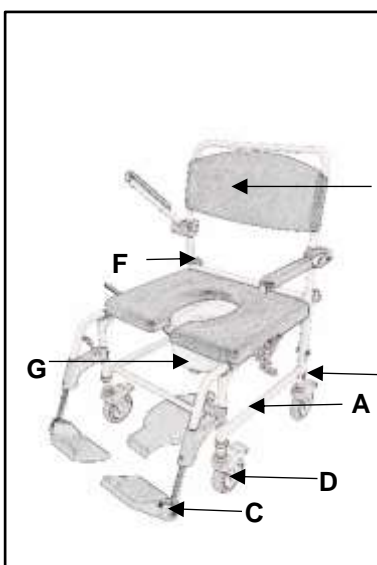

Inside the box you should find:

1 x Chair Base Frame	A
1 x Back Rest	B
2 x Footrests	C
4 x Braked Castors(T210 / MT210)	D
2 x Braked Castors & 2 x 685mm Wheels (T212 / MT212)	
4 x Castor Pin Clips	E
2 x Hand-wheels	F
1 x Commode Potty	G
2 x Wheel Axles, Axle bushes & Stabilising bars (T212 / MT212)	
2 x Brake Assemblies & Brake Brackets. (T212 / MT212)	
1 x User Instructions	-
1 x Assembly Instructions	-

Assembly Instructions

Please note:

- When unpacking carefully examine the product to ensure it is complete. See contents list above.
- Separate Assembly instructions are included in the box, but are also included here.
- Before you begin assembly check for any damage. If damaged please contact your supplier and refrain from using the product.
- **Do not attempt to use if the product is damaged.**

Assembly:

- Place the Chair Base Frame (A) on a level floor. At the rear of the frame you will see two black threaded hand wheels (F). Undo the wheels so that the holes are clear.
- Place the two legs of the back rest (B) into the holes on the back edge of the base frame (A).
- Secure the back rest by inserting the hand wheels and turning until you feel the nut biting against the leg of the back. **Do not overtighten – finger tight is sufficient.**
- Place all four castors into the leg tubes and secure using the pin clips. If fitting T212/MT212 see below.
- On each side of the front legs of the commode main frame there are two retaining lugs either side.
- Take the footrests and align the holes in the inverted “U” section with the lugs. Lower down into place.
- Rotate the foot rest towards the commode so that they clip into place.
- They can be rotated up and down as necessary, should they temporarily not be required.

For T212 / MT212:

- Instead of fitting the two rear castors, replace with the stabilising bars.
- Fit the axle bushes through the locating holes in the frame from the outside and fit the nut on the inside, now place the axle from the outside of the wheel (black knob outside) and push through the axle until it clicks into place.

User Instructions for the Attendant Propelled & Self Propelled Shower Commode Chairs range – T210, T212, MT210 & MT212

- Fit the brake bracket and brake onto the frame and tighten the two Allen screws slightly, ensure that the brake works then fully tighten the Allen screws.

Correct Use

- Empty the potty after each use.

Castors:

- To lock: push down the lever on the castor body (all 4 are lockable).
- To unlock: push the lever in the opposite way.
- Insert the modesty cover for showering use.

Accessories

A Lap Strap is available for use with the chair. This is available as a separate item from Cefndy Healthcare. It is **essential** that a risk assessment is carried out for each individual user of the Lap Strap. Please refer to the User Instructions supplied with the strap.



Cleaning

- The Product can be cleaned using a mild detergent and a mild disinfectant or antibacterial solution caution should be taken when using cleaning agents/chemicals in understanding their suitability.
- Do not use an abrasive cleaner as this may damage the frame and invalidate The frame is suitable for mechanical cleaning/disinfection in industrial cleaning machines up to 80°C. Thorough drying of the chair is necessary afterwards.

Maintenance Requirements

- The product should be checked on a regular – this should include the items which can be assembled/ disassembled to ensure their fit and location is as intended, i.e.; The Back rest is correctly seated and hand wheels secure etc.
- Please ensure the product is dried following Cleaning.

Warning

- We recommend a full risk assessment is carried out before use. This should include consideration for the equipment's intended use, the capacity of the user to safely use the equipment, the environment in which the equipment is intended to be used and **all** other relevant considerations.
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.

Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects, damage incurred through misuse is not covered. This warranty does not affect your statutory rights. Do not use in the event of fault or damage.

Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to minimise the risk of infection.
- Materials used are essentially non-hazardous and can be disposed of appropriately consideration should be given to local re-cycling arrangements.

Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 0044 1745 343877
E-mail: cefndy.sales@denbighshire.gov.uk