

User Instruction for the Wide Foot Stool (Brown) X700



1. Introduction

The X700 is a complementary product to the Bariatric High Back Chair. Its easy clean upholstery provides for longevity of service and the top is made from high density deep foam for ultimate comfort. The none-marking rubber legs with give you secure positioning. This is designed for the use of supporting legs whilst sitting in a chair – not for sitting on or any other use.

Product Dimensions	
Upholstery length	480mm
Upholstery width	440mm
Upholstery depth	120mm
Stool height (from-to)	430mm-500mm

2. Contents of the Box

- Wide Footstool (X700)
- Extended Legs & Pin Clips
- User Instructions (this leaflet)

3. Assembly Instructions


- Carefully unpack and examine the product to ensure it is complete.
- Before use check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product. **DO NOT ATTEMPT TO USE IF PRODUCT IS DAMAGED.**

4. Correct Use

Height Adjustment:

- Remove the pin clip from the leg.
- Move the extended leg piece up or down to the required location (adjustable in increments of 25mm).
- Push the pin clip back in ensuring it firmly locates in both the leg and the extended leg piece.

User Instruction for the Wide Foot Stool (Brown) X700

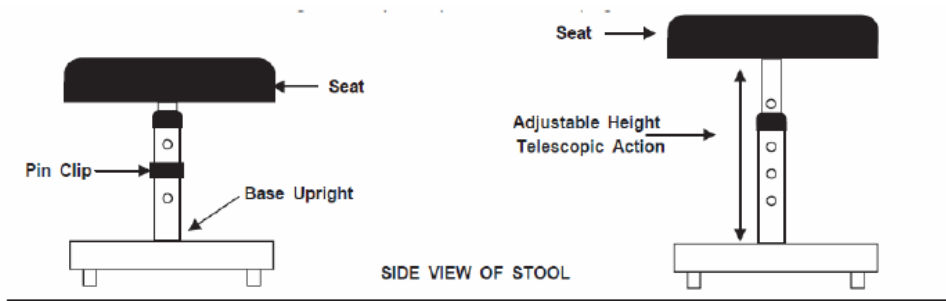

5. Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this will damage the frame and nullify your warranty.
- This product is **not** suitable for power/jet washing and industrial washing machines.

6. Maintenance Requirements

The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

7. Warning

- This product is intended for users of a maximum of 157kg/25stone in weight.
- We recommend a full risk assessment should be carried out before use, to ensure appropriate user and environment requirements.

8. Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

9. Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection
- Materials used are essentially non hazardous and can be disposed of appropriately

10. Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:
Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 0044 1745 343877 (Quality Assurance Dept)
E-mail: cefndyengineering@denbighshire.gov.uk