

This product can be purchased in various configurations as detailed in the table. There are also 3 options in width between the arms with are identified by the prefix in italics at the beginning of the product code, e.g. M152/15.

Product code & description	
152/15	Adjustable Height
153/15	Adjustable Height / Floor Fixing Feet

Mediatric™ – 600mm (product code begins with “M”)

Bariatric – 700mm (product code begins with “X”)

Super Bariatric – 800mm (product code begins with “SX”)

Seat height range = 450mm – 600mm

1. Introduction

The Raised Toilet Seat Range is designed with hygiene in mind; the seat clips off the frame to aid cleaning. The Raised Toilet Seat is adjustable in height at 25mm intervals. The feet are non-marking & non-slip for extra user safety (152 versions only). 153 version are designed to be securely fixed to the floor.

All structural joints are fully welded. As well as providing structural strength this feature also aids product hygiene by reducing the possibility of bacterial entrapment. The steel framework is also coated in an anti-bacterial powder coated finish.



Version pictured is an M152/15

Product specification	
Seat depth	460 mm
Max footprint “M” range	660mm x 600mm
Max footprint “X” range	760mm x 600mm
Max footprint “SX” range	860mm x 600mm
Seat to top of arms	170mm
Maximum user weight	318 KG (50 Stone)


Parts list	
Description	Part No
Crutch tip	74G3
Extension leg (standard)	56C319P
Extension leg (transporter type kit)	63YESSP/I
Floor fixing seat set	T176I/SET
Pin clip	63PC
Anti-rattle collar	74GAR/2
Plastic Sleeve	71076
Clip on seat “M” range	720600TS
Clip on seat “X & SX” range	720700TS

2. Contents

- Commode
- User Instructions (this leaflet)
- Batch label – This will be found on the product and will include the product code as well as a number. In the event of needing to contact the manufacturer about this product always quote the Batch Number.

3. Assembly Instructions

- Carefully unpack and examine the product to ensure it is complete.
- Before use check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product.
- **DO NOT ATTEMPT TO USE IF PRODUCT IS DAMAGED.**
- To adjust the Raised Toilet Seat height (standard leg or Floor Fixed version)
 1. Remove the pin clip from the leg individually
 2. Select the height by positioning each extension leg individually aligning the holes in the extension leg and frame
 3. When the required height is selected refit the pin clip into the appropriate hole
 4. Repeat the above on the other three legs
 5. Ensuring all four pin clips are fully engaged and check that all four legs are set to the same height

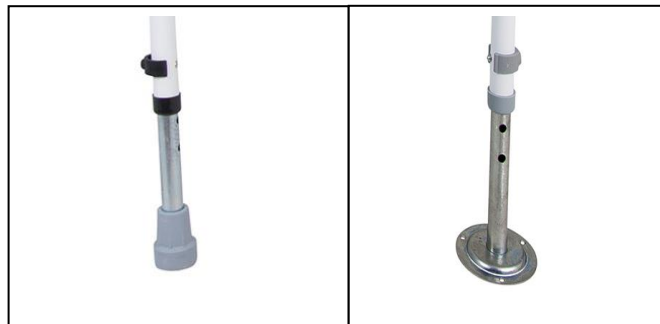

- We are unable to advise on the best method of securing the Floor fixed Foot to the sub-floor as this may vary dependent on the materials/structure of the sub-floor, if in doubt consult a competent person.
- The product must be set to the required height before the foot is secured to the floor (as above).

5. Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this will damage the frame and nullify your warranty.
- It is not recommended that polyurethane is cleaned using power/jet washing or industrial washing machines.



User Instruction for the 2015 Mediatric/Bariatric & Super Bariatric Raised Toilet Seat range

Use of power/jet washing & industrial washing machines (Not recommended for PU products):

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that frames are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the framework as it could cause permanent rusting of the frame (leave frames open in a warm environment for several hours to ensure all internal moisture evaporates).
- We cannot offer advice on the use of industrial washing machine to clean this product

6. Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

7. Warning

- This product is intended for users up to a maximum of 318kg/50stone in weight.
- We recommend a full risk assessment be carried out before use.
- Under no circumstance should the Raised Toilet Seat be moved whilst occupied.
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.

8. Warranty

This product carries a 12 month warranty from the date of purchase from Cefndy against manufacturing defects. Misuse or accidental damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage **however we can advise that this product has been designed, manufactured and independently tested in the UK (including cyclic testing to represent real life usage).**

9. Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non-hazardous and can be disposed of appropriately.

10. Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare, Cefndy Road. Rhyl, Denbighshire, LL18 2HG
Tel: 0044 1745 343877 (Quality Assurance Dept)
E-mail: cefndy.sales@denbighshire.gov.uk